

Speech Assistant AAC Set-Up Instructions (Apple)



Speech Assistant AAC is a simple & affordable communication app that can be loaded onto your personal phone or tablet. The app allows users to type & "speak" messages on the device's keyboard OR choose from a large number of customizable saved messages. The app uses your device's own keyboard and text-to-speech engine, making this a great option for speakers of other languages including English, French, Spanish, Italian, Cantonese, Russian and more!

Step 1: DOWNLOAD THE APP

Download the app 'Speech Assistant AAC' from iTunes App Store.



Apple users can purchase the full version of the App for \$27.99 CAD from the App Store.

Step 2: CUSTOMIZING THE APP

The app will come pre-programmed with generic categories and messages. Feel free to keep and edit the existing categories & messages, or simply delete them and start from scratch.

Customizing your Categories:

- 1. Go to Settings menu (the gear icon)
- 2. Select 'Categories & Phrases'
- 3. Select 'Categories'
- 4. Select the '+' icon to add a new category
- 5. Select a category, then the pencil to edit the category.
- 6. Select a category, then 'delete' to delete the entire category
- 7. Save & go back to test your new category

Customizing your Phrases:

- 1. Go to Settings menu (the gear icon)
- 2. Select 'Categories & Phrases'
- 3. Select 'Categories'
- 4. Select the Category you wish to edit or add to
- 5. Select the '+' icon to add a new phrase
- 6. Select a phrase, then the pencil to edit the phrase
- 7. Select a phrase, then 'delete' to delete the entire phrase
- 8. Save and go back to test your new phrase

Step 3: USING SPEECH ASSISTANT AAC

Using the Keyboard to type a Message:

- 1. Tap anywhere in the white message window at the top of the screen.
- 2. A keyboard will pop up at the bottom of the screen.
- 3. Type in your message.
- 4. Press the "speak" button to play the message out loud.
- 5. Press the "clear" button to clear the message window.

Using Saved Messages:

- 1. Select a Category
- 2. Select a Phrase
- 3. Select the 'clear' button to clear the text from the message window at the top of the screen

Visit the website for more information: https://www.asoft.nl/