

Speech Assistant AAC Set-Up Instructions (Android)



Speech Assistant AAC is a simple & affordable communication app that can be loaded onto your personal phone or tablet. The app allows users to type & “speak” messages on the device’s keyboard OR choose from a large number of customizable saved messages. The app uses your device’s own keyboard and text-to-speech engine, making this a great option for speakers of other languages including English, French, Spanish, Italian, Cantonese, Russian and more!

Step 1: DOWNLOAD THE APP

Download the app '**Speech Assistant AAC**' from Google Play Store



Android users can download a FREE version of this app from the Google Play Store, OR for more customization, purchase the full version for \$7.50 CAD.

Step 2: CUSTOMIZING THE APP

The app will come pre-programmed with generic categories and messages. Feel free to keep and edit the existing categories & messages, or simply delete them and start from scratch.

Customizing your Categories:

1. Open the App
2. Go to Settings menu
3. Select 'Categories & Phrases'
4. Select 'Categories'
5. Select the '+' icon to add a new category
6. Select the 'X' to delete a category
7. Select 'save'

Customizing your Phrases:

1. Open the App
2. Go to Settings menu
3. Select 'Categories & Phrases'
4. Select 'Phrases'
5. Select the Category you wish to edit
6. Select the '+' icon to add a message
7. Select the red 'X' to delete a message
8. Select 'save'

Step 3: USING SPEECH ASSISTANT AAC

Using the Keyboard to type a Message:

1. Tap anywhere in the white message window at the top of the screen.
2. A keyboard will pop up at the bottom of the screen.
3. Type in your message.
4. Press the "speak" button to play the message out loud.
5. Press the “clear” button to clear the message window.

Using Saved Messages:

1. Select a Category
2. Select a Phrase
3. Select the 'clear' button to clear the text from the message window at the top of the screen

Visit the website for more information: <https://www.asoft.nl/>